User manual
Bedienungsanleitung
Gebruiksaanwijzing
Manuel d'utilisation
Manual del usuario
Manual do utilizador
Manuale d'uso
Bruksanvisning
Brugsanvisning

Looky logo is a trademark of Rehan Electronics

First Edition (July 2006) V3.3

English	4	En
Deutsch	10	De
Nederlands	16	Ne
Français	22	Fr
Español	28	Es
Português.....	34	Po
Italiono	40	It
Svenska	46	Sv
Dansk	52	Da

En Congratulations with the purchase of your Looky. On the last page of this manual you will find a (product-) control-summary which will be referred to in this manual. Read the manual carefully before you start using the Looky.

Safety and Maintenance:

Protect the device and its batteries from moisture, rain, sand or extreme temperatures like radiators or direct sunlight.

Do not use cleaning products that contain alcohol, methylated spirits, ammonia or abrasives. These can damage the device.

Only use accessories that the producer supplies you with or approves of.

Never connect the Looky to the adapter when the Looky is switched on.

Never throw batteries into a fire. Batteries that are defect and chargers are considered chemical waste. They therefore need to be disposed of at a refuse/recycling centre of your choice.

Installing the Looky:

Place the batteries in the battery compartment (3). Pay attention to the polarity of the batteries and ensure that the nylon string (4) does not make contact with the exposed terminal(s) of the batteries.

Charge the batteries completely before using the Looky. Carefully read the charging instructions!
--

Instructions for charging:

Put the adaptor in the wall socket and connect its plug to the adaptor entrance (5) of the Looky. Two short beeps will be heard.

LED (6) burning constantly	= batteries are being charged
LED (6) blinking	= batteries are fully charged
LED (6) blinking	= batteries are defect or
+ audible dual signal	are non-rechargeable batteries

The charging process will be interrupted if you turn on the Looky!

Pay attention:

- * Charge the batteries in a well ventilated environment. (Always remove the Looky from its carrying case. Do not place it in direct sunlight or near a radiator.)

Battery maintenance:

- * Remove the batteries from the Looky if you do not intend to use it for an extended period of time. This will improve the longevity of the batteries.
- * Batteries are sensitive to overcharging. It is recommended to only charge the batteries when they are completely drained.
- * Chargeable NiMh batteries have an auto discharge that strongly depends on the temperature of the environment. (e.g. never leave your batteries in a car that is parked in direct sunlight.)
- * New NiMh batteries must be charged several times before reaching their nominal capacity.
- * Charge NiMh batteries at least once every 6 months.
- * The average lifespan of a battery is usually 300 charging cycles, but this depends on the use. Total capacity will slowly decrease after multiple charging cycles.

Switching the Looky on or off:

Keep one of the buttons (1,2) pressed (ca. 3 seconds) to switch the Looky on. A two-pitched signal will be heard.

When you have the Looky switched on, and it doesn't detect any movement on the screen, it will switch itself off after 3 minutes. A two-pitched signal will be heard.


When the batteries are empty, the Looky will switch itself off. You will hear a high pitched tone that fades away.

Folding handle:


The Looky has a foldable handle. This handle can be used in several positions. When the handle is extended, you can use the Looky as a magnifier. The closer you get to the object, the bigger the magnification will be (from 3x to 8x).

It is also possible to use the Looky with the handle folded in. The magnification will then be about 8x. This is easier when you want some stability during the reading of texts.

If you want to alter the magnification, but are experiencing some problems with stability, we advise you to use the Looky Support. This is a convenient tool to change the magnification and viewing angle while keeping the stability.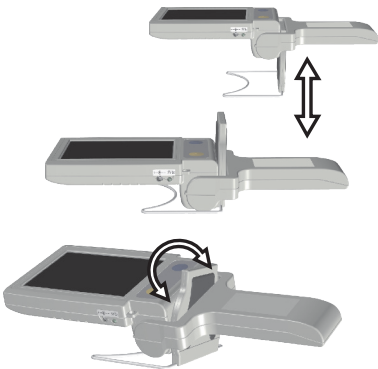

It is also possible to use the Looky when writing. To do this, the handle must be extended to the maximum. When you then press the handle flat to the table, the display will rise up at an angle. The space that is created underneath the Looky can be used to write. We advise to use the colour mode when writing.

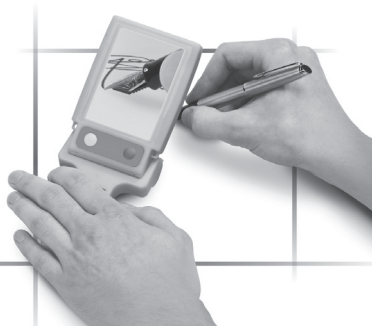Display (2):

By pressing the blue button you can select 3 display options:

- Colour
- Positive high contrast (black text on a white background)
- Negative high contrast (white text on a black background)

Freeze function (1):

By pressing the yellow button you can freeze the image on the Looky's display. Press the button again to revert to the regular viewing mode. You can recognize this button by the elevated dot in the middle of the button.

When the image is frozen, the lighting will be dimmed to save battery capacity.

It is possible to switch between the 3 display modes when the image is frozen.

Troubleshooting:

If the following table does not offer a solution to your problem, please contact your distributor.

Problem		Solution
1	No Picture, dark or blurry image.	- Switch the Looky on (check if the lighting is on). - Image is frozen, press the yellow button once. - Move the Looky closer to the object to get a sharper image. - Switch the viewing mode to colour.
2	Looky cannot be switched on.	- Keep one of the buttons pressed for an extended period of time. Repeat several times if necessary. - Charge the batteries. - Place and / or check the polarity of the batteries. - Check if the battery contacts are sufficiently clean.
3	Looky switches itself off within minutes, sounding a dual-pitched tone.	- The Looky switches itself off if it cannot detect movement on the screen for 3 minutes.
4	Looky switches itself off within minutes with a single high pitched tone that fades away.	- Charge the batteries. - Check the battery type (NiMh). Alkaline or other non-chargeable batteries can severely shorten the usage. - Check if the battery contacts are sufficiently clean. - Replace with new batteries with a capacity $\geq 2500\text{mAh}$.
5	Looky switches off prematurely.	- Check the battery type (NiMh). Alkaline or other non-chargeable batteries can severely shorten the usage. - There are many variables that can affect the battery capacity negatively. In any case, use batteries with a capacity $\geq 2500\text{mAh}$.

Specifications:

Measurements (length x width x height) and weight (excluding batteries) :

- * Looky : 110 x 75 x 32 (mm)(140grams)
- * Complete package : 480 grams

Display:

- * TFT : 3,5"

Magnification:

- * minimum : 3x
- * maximum : 8x

Power supply:

- * Batteries : 2x AA/LR6¹
- * Adaptor : 5V DC 2A

Conditions of use:

- * Temperature : +5°C tot +35°C
- * Air humidity : <70%, no condensation

Time of use and charging time:

- * You can use the Looky for 1,5 to 2 hours on average at the recommended capacity².
- * The average charging cycle lasts 6 hours, depending of the quality and capacity of the battery.

¹ Recommended: Chargeable batteries NiMH batteries with a capacity $\geq 2500\text{mAh}$.

² Time of use depends on many variables and can fluctuate.

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Rehan Electronics Ltd.