Looky<sup>5</sup>

User Manual

The Looky 5 logo is a trademark of  
Rehan Electronics Ltd.  
User manual V1.5

English .....	4
---------------	---

English

Nederlands .....	21
------------------	----

Nederlands

Deutsch .....	38
---------------	----

Deutsch

Français .....	55
----------------	----

Français

Italiano.....	72
---------------	----

Italiano

Español .....	89
---------------	----

Español

---

## Table of contents


1. Introduction .....	5
2. General description Looky 5 .....	5
3. Intended use.....	6
4. Safety and Maintenance .....	6
5. Installing the Looky 5 .....	7
6. Instructions for charging.....	7
7. Battery handle maintenance.....	8
8. Folding handle .....	8
9. Switching the Looky 5 on or off .....	9
10. Focusing the Looky 5 .....	9
11. Far View function .....	9
12. Switching off the LED.....	9
13. Display modes.....	10
14. Zooming.....	10
15. Freeze function.....	11
16. Magnetic lines .....	11
17. Looky 5 menu .....	12
17.1 Sound, brightness and reading lines.....	12
17.1.1 Sound icon .....	12
17.1.2 Brightness icon.....	12
17.1.3 Reading lines .....	13
17.2 Display modes and battery.....	13
17.2.1 Battery icon .....	13
17.2.2 Display mode icon .....	14
18. Picture output .....	14
19. Writing with the Looky 5 .....	14
20. Hidden menu access .....	15
21. Troubleshooting.....	18
22. Specifications .....	19
23. Warnings .....	20
24. Discarding the Looky 5 or battery .....	20
25. Warranty regulations .....	20
26. Regulations .....	20

---

## 1. Introduction

Congratulations with the purchase of your Looky5! In this manual you will find a detailed description of all the functions of the Looky5. Read this manual carefully before you start using the Looky5. If you still have any questions after reading this manual, please contact your supplier.

## 2. General description Looky 5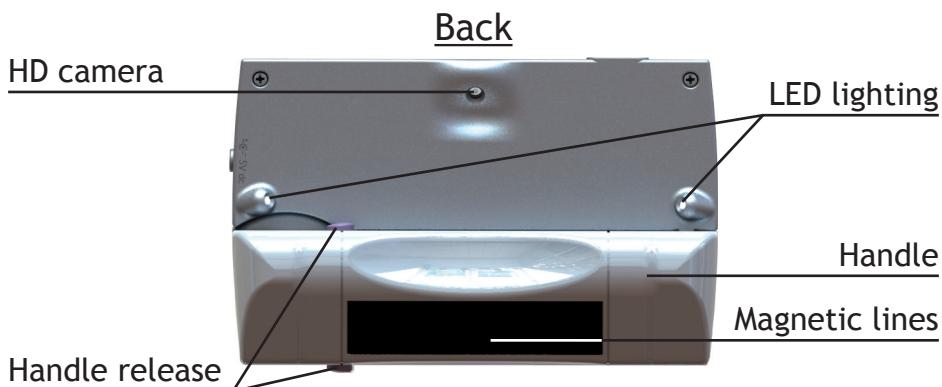

### 3. Intended use

This device is solely used for displaying enlarged representations of text, photographic material and objects. When the Looky 5 is used for any purpose other than the intended use, the CE regulation will not apply.

**Caution:** do not, under any circumstances, perform maintenance or repairs on the device; always ask a manufacturer approved technician to do this. Warranty will be voided if the Looky 5 is attempted to be repaired by anyone other than a technician approved by the manufacturer.


### 4. Safety and Maintenance

Before use, be sure to comply to with the following safety and maintenance instructions:

- Protect the device and its batteries from moisture, rain, sand or extreme temperatures like radiators or direct sunlight.
- Do not use cleaning products that contain alcohol, methylated spirits, ammonia or abrasives. These can damage the device or its touch screen.
- Only use accessories that the manufacturer supplies you with, or approves of.
- Never throw batteries into a fire. Batteries and chargers that are defect are considered chemical waste. They therefore need to be disposed of at a refuse/recycling centre of your choice.
- Note that the use of a screen protector might influence the functioning of the touch screen.

## 5. Installing the Looky 5

Connect the battery handle of the Looky 5 by sliding it onto the connector until you hear it clicking into place. This battery handle also functions as the Looky5's handle.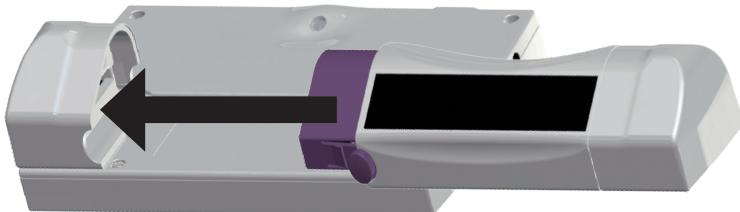

To replace the battery handle push the 2 purple battery release buttons on the side of the battery handle, and slide on the replacement battery handle.

Charge the battery handle completely before using the Looky5.  
Please read the charging instructions carefully.

## 6. Instructions for charging

Put the adaptor in the wall socket and connect its plug to the charging adaptor port of the Looky5.

LED burning constantly  
LED off  
LED blinking

batteries are being charged  
batteries are fully charged  
error / batteries are defect

Charge the batteries in a well ventilated environment. Remove the Looky 5 from its carrying case when charging. Don't place it in direct sunlight or near a radiator whilst charging.

When the battery is empty, the Looky 5 will switch itself off. You will hear a high pitched tone that fades away.

## 7. Battery handle maintenance

- Remove the battery handle from the Looky 5 if you do not intend to use it for an extended period of time. This will improve the longevity of the battery handle.
- Batteries are sensitive to overcharging. It is recommended to only charge the battery handle when it is completely drained.
- Chargeable Lithium-Ion batteries have an auto discharge that strongly depends on the temperature of the environment (e.g. never leave the battery handle in a car that is parked in direct sunlight).
- New Lithium-Ion batteries must be charged several times before reaching their nominal capacity.
- Charge Lithium-Ion batteries at least once every 6 months.
- The average lifespan of the battery handle is usually 300 charging cycles, but this depends on the use. Total capacity will slowly decrease after multiple charging cycles.

## 8. Folding handle

The Looky 5 comes with a replaceable foldable handle with a built-in battery. This handle can be used in several positions.

When the handle is extended, you can use the Looky 5 as a magnifier. The closer you get to the object, the bigger the magnification will be. You can also hold the Looky 5 up to see distant objects as the Looky 5 can focus from 5cm to infinity.

It is also possible to use the Looky 5 with the handle folded in. You can easily set the device on the table and it will automatically stand at an angle. The magnification will then be about 8x. This is easier when you want some stability during the reading of texts.

---

## 9. Switching the Looky 5 on or off

To switch the Looky 5 on, press the ON/OFF button once. You will hear a scaled sound increasing in pitch.

To switch the Looky 5 off, press the ON/OFF button for 3 seconds. You will hear a scaled sound decreasing in pitch.

When the Looky 5 is switched on, and it doesn't detect any movement on the screen, it will switch itself off after 3 minutes. A two-pitched signal will be heard.

## 10. Focusing the Looky 5

The Looky 5 uses a push-to-focus method to focus the image on the screen. Simply touch the screen on the section where you want the Looky 5 to focus; the camera will focus on that area.

## 11. Far View function

The camera of the Looky 5 can focus on objects from 5 cm. distance to infinity. Just hold the device up to view the object. Remember to touch the screen in order to focus the camera on the object you are viewing.



## 12. Switching off the LED

By touching the centre of the screen for 1 second you can switch off the LED at the back of the Looky5. You will hear a beeping sound indicating that the LED is switched off.

Touch the centre of the screen again for 1 second to turn the LED back on. Another beeping sound will be heard.

## 13. Display modes

By swiping from the top to the bottom of the screen, or vice versa you can cycle through multiple colour modes.

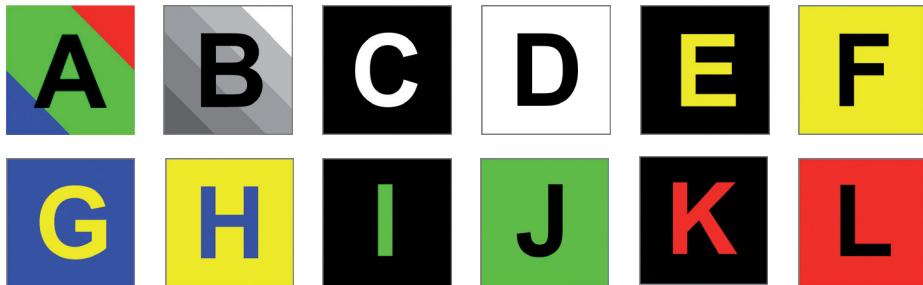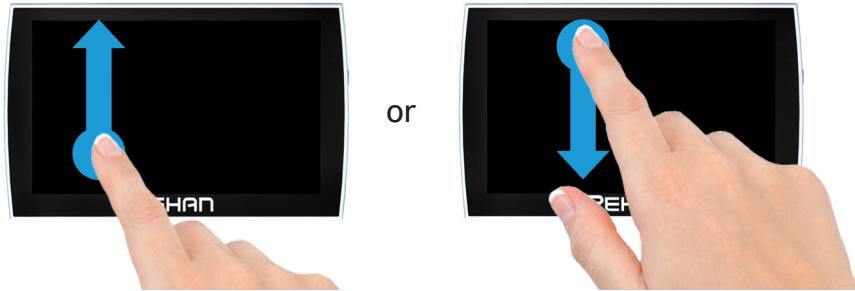## 14. Zooming

To alter the magnification you slide your finger up/down on the left or right hand side of the touch screen.

To zoom in slide your finger upwards on the zooming icon, to zoom out, slide downwards on the zooming icon. The magnification range of the Looky 5 ranges from 2x to a 20x magnification.

## 15. Freeze function

By shortly pressing the ON/OFF button you can freeze the image on the Looky5's display. Press the button again to revert to the regular viewing mode.

When the image is frozen, the lighting will be dimmed to save battery capacity.

When the image is frozen, you can use 1 finger to pan through the frozen image.

All functions of the Looky 5 can still be used while the image is frozen.

## 16. Magnetic lines

On the bottom side of the handle, there is a magnetic area that can be used with the magnetic sheet.

Slide the sheet underneath the text you want to read. Now place the Looky 5 on the text and slide it across the magnetic sheet. The ridges in the magnetic sheet will guide the Looky 5 along the lines; making it easier to keep the Looky 5 straight while reading.

## 17. Looky 5 menu

### 17.1 Sound, brightness and reading lines

Use 1 finger to touch the bottom section of the screen for 2 seconds to generate the sound menu, brightness menu and the reading line option in the bottom half of the screen. These settings will be saved when the Looky 5 is switched off.

#### 17.1.1 Sound icon

With this icon you can switch the sound on or off.


sound on

sound off

#### 17.1.2 Brightness icon

With this icon you can set the brightness of the screen. There are 5 different intensity settings.

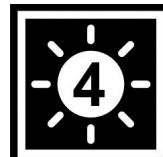least brightness

most brightness


### 17.1.3 Reading lines

By tapping this icon you can generate a horizontal or vertical reading line on the screen.


## 17.2 Display modes and battery

Use 1 finger to touch the top section of the screen for 2 seconds to generate the battery and display mode menu in the top half of the screen.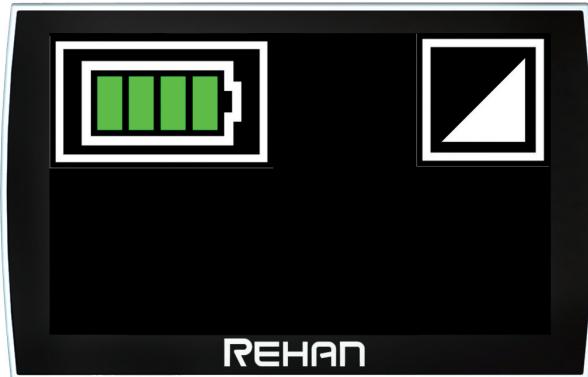

### 17.2.1 Battery icon

This icon indicates the remaining battery life. There are 5 different icons to indicate this.

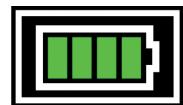
battery empty

battery full



When the battery is charging, the menu will display the charging icon.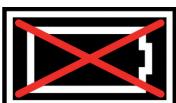

When the adaptor is connected, but the battery is not connected, this icon will appear.

## 17.2.2 Display mode icon


By touching the display mode icon, an overview of all available display modes will appear. You can easily select which display mode you want to use.

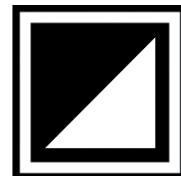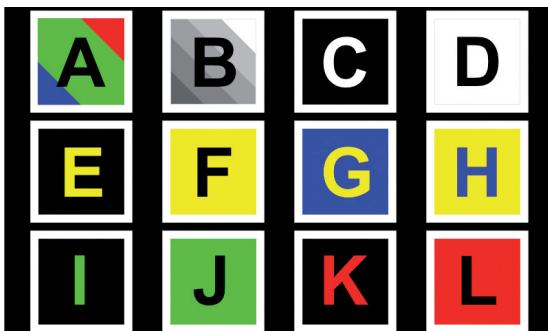
## 18. Picture output

The Looky 5 can output the picture to an external display. Do this simply by connecting the appropriate cable to the HDTV port or the Composite Video port.

Be sure to connect the Looky 5 to an earthed TV for the best results.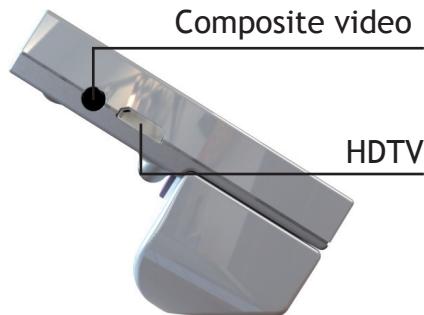

## 19. Writing with the Looky 5

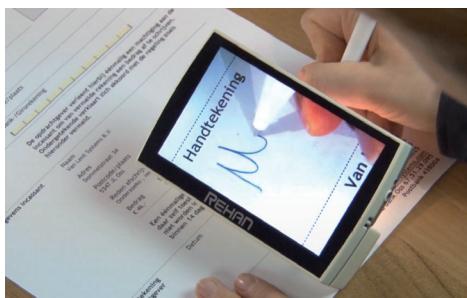Fold the handle inwards and place it on the paper you want to sign.

## 20. Hidden menu access

The Looky 5 has a hidden menu consisting out of 6 sections where you can customize certain aspects of the Looky 5 software to your liking.

To enter the hidden menu keep the On/Off button pressed for at least 4 seconds.


You can scroll through each section by swiping from top to bottom. Swipe from right to left to select other options.

To exit the hidden menu, press the On/Off button once.

### 20.1 Enabling/Disabling colour modes

By enabling or disabling colour modes in this section of the hidden menu you can determine which colour modes are available during basic use of the Looky 5.

Please note that, regardless of which colour modes have been disabled, all colour modes can still be accessed via the Looky 5 menu (see chapter 17.2.2 for more information).

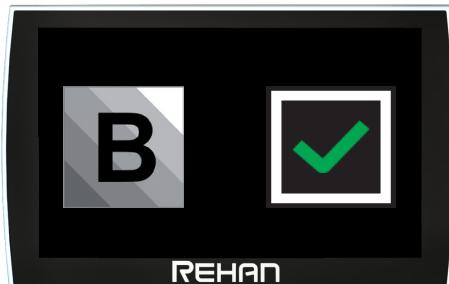Tap the tick box to disable the selected colour mode.


Tap the cross box to re-enable the selected colour mode.

**Note:** the full colour mode cannot be disabled.

## 20.2 Refresh Rate

In this section you can change the refresh rate of the TFT.

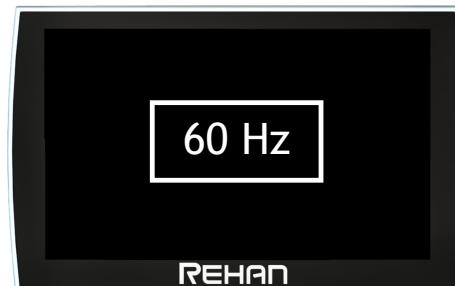## 20.3 User interface option

The Looky 5 has 3 user interface options that the user can choose from: standard, basic or basic continuous. The standard user interface is explained in chapter 15.

The basic user interface only contains the zoom, display mode and freeze image functions.

The Basic Continuous user interface is the same as the basic user interface with the exception that it doesn't disappear automatically when the screen hasn't been operated in 2 seconds. Combined with the included masking sticker this can create an easier to use interface.

Standard

Basic

Basic continuous


### Applying the masking sticker

Take the supplied masking sticker from the backing sheet and place it with the rounded side to the right on the right side of the screen so that it exactly surrounds the buttons of the basic continuous-user interface.


## 20.4 Enable/Disable 3 minute auto shutdown

When the Looky 5 does not detect movement for 3 minutes, it automatically shuts itself down to preserve battery capacity. This feature can be switched off in the hidden menu.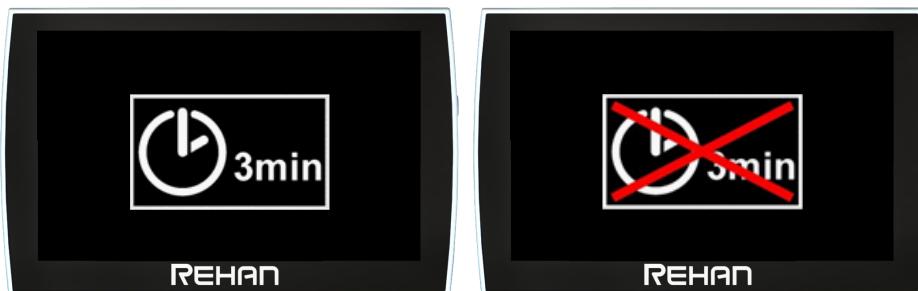

## 20.5 Enable/Disable reading lines

In this section you can choose whether to enable/disable the horizontal or vertical reading lines. Swipe from right to left to switch between the horizontal and vertical reading lines.

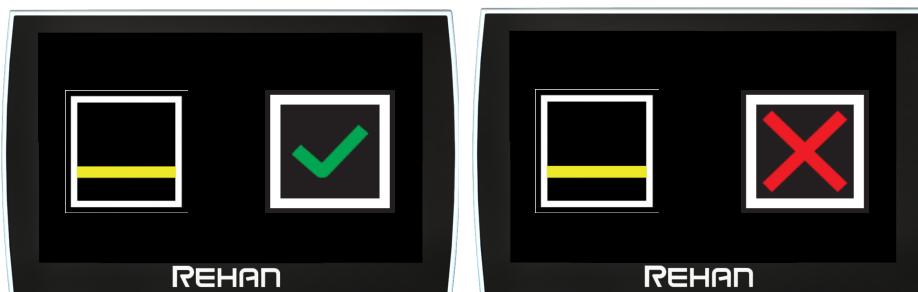Tap the tick box to disable the horizontal reading line.

Tap the cross box to re-enable the horizontal reading line.

## 20.6 Software information

This section shows the software version of the Looky 5.

## 21. Troubleshooting

If the following table does not offer a solution to your problem, please contact your distributor.

Problem	Solution
1 No picture, dark or blurry image.	<ul style="list-style-type: none"><li>• Switch the Looky 5 on.</li><li>• Image is frozen, shortly press the ON/OFFbutton once.</li><li>• Switch the viewing mode to colour.</li></ul>
2 Looky 5 cannot be switched on.	<ul style="list-style-type: none"><li>• Press the ON/OFF button to switch the Looky 5 on.</li><li>• Charge the battery handle.</li><li>• Reinsert or replace the battery handle.</li></ul>
3 Looky 5 switches itself off within minutes, sounding a dual-pitched tone.	<ul style="list-style-type: none"><li>• The Looky 5 switches itself off if it does not detect movement on the screen for 3 minutes.</li></ul>
4 Looky 5 switches itself off within minutes with a single high pitched tone that fades away.	<ul style="list-style-type: none"><li>• Charge the battery handle.</li><li>• Replace the battery handle with a new battery handle.</li></ul>

## 22. Specifications

### Main specification

Measurements	131 × 75 × 32 mm (l × w × h)
Weight	300 grams (including battery)

### Display

TFT diameter	5.0"
Other specifications	High Brightness Touch technology

### Camera

Minimum magnification	2×
Maximum magnification	20×
Focus	5cm - infinity

### Power supply

Battery system	Interchangable (inside battery handle) Lithium-Ion 4000mAh
Adaptor	5VDC/1.8A

### Conditions of use

Temperature	+5 °C till +35 °C / 50 °F till 95 °F
Air humidity	<70% (no condensation)

### Time of use and charging time

Time of use	Up to 3 hours
Charging time	approx. 5 hours

### Package contents


Looky5
Battery handle (4000mAh)
Charging adapter
Travel pouch
Magnetic sheet
User manual
HDTV cable
SDTV cable
Cleaning cloth

## 23. Warnings

This icon indicates that users with pacemakers should take care with the magnetic area of the handle on the Looky 5 as this might, depending on your pacemaker, interfere with the pacemaker. Please contact your surgeon to inform yourself on the effects of magnets with your pacemaker.

## 24. Discarding the Looky 5 or battery

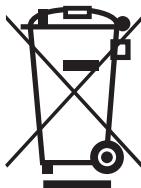This symbol of the crossed out wheelie bin indicates that the product must not be disposed of with your other household waste. Check local regulations for proper disposal of electronic products.

## 25. Warranty regulations

This device is subject to the warranty regulations of your supplier. For further information regarding these regulations, please contact your supplier.

## 26. Regulations

Products with CE marking comply with EMC Directive (2004/108/EC), and the Low Voltage Directive (2006/95/EC) issued by the Commission of the European Community.

Compliance with these directives implies conformity to the following standards:

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REHAN ELECTRONICS LTD.