
i-See HD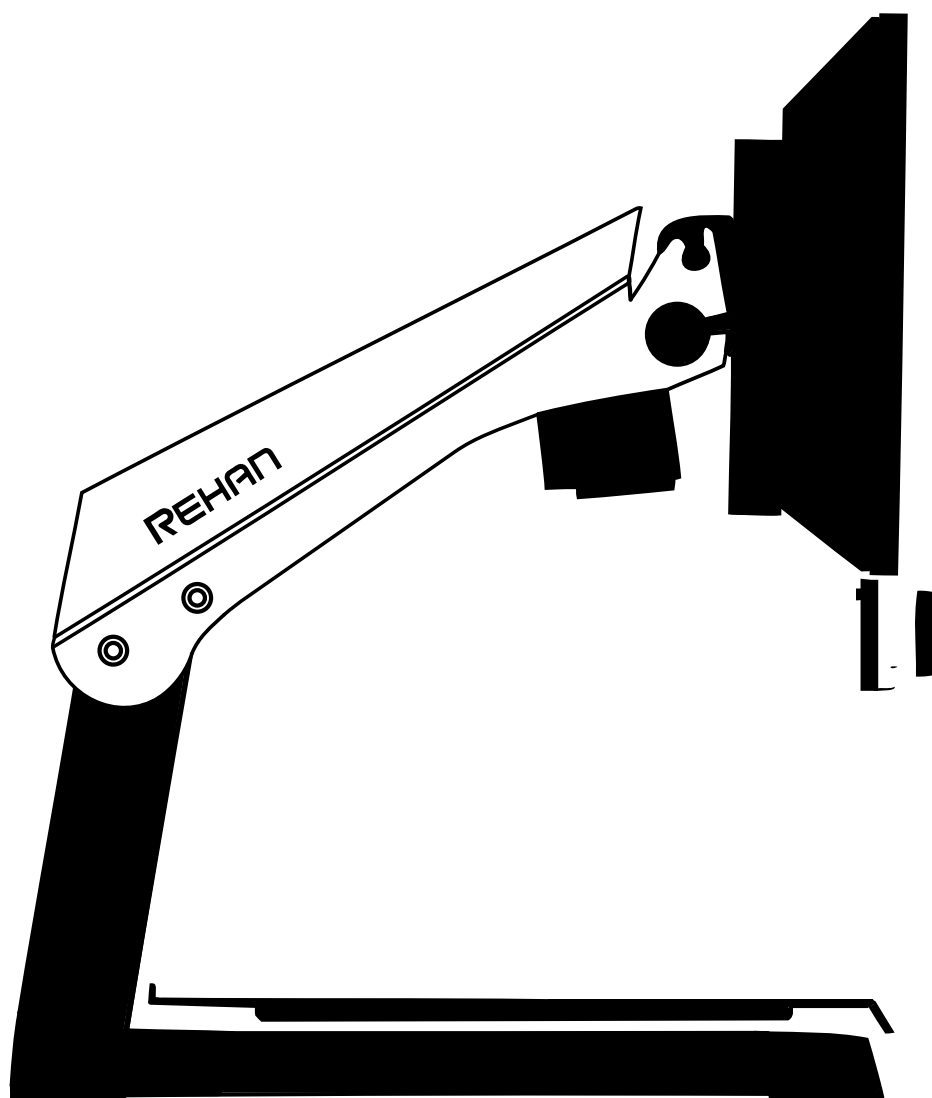

GEBRUIKSINSTRUCTIES
INSTRUCTIONS FOR USE
GEBRAUCHSANLEITUNG
ISTRUZIONI D'USO
MANUAL DE INSTRUCCIONES
MANUEL DE L'UTILISATEUR

Nederlands4

NL

English13

EN

Deutsch.....22

DE

Français31

FR

Español40

ES

Italiano49

IT

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Read this manual carefully before using the i-See HD.
Keep the user manual for future reference.

EN

Table of contents

Foreword 14

1 Introduction..... 14

2 Package contents 14

3 Safety..... 15

3.1 Transporting the i-See HD 15

4 Installation..... 15

4.1 Attach the TFT screen..... 15

4.2 Connecting the cables 16

4.3 Tilt the screen 16

4.4 Adjust the height of the screen 16

5 Controls..... 17

5.1 On/off switch..... 17

5.2 Display modes 17

5.3 Zooming in and out..... 17

5.4 Zoom presets..... 17

5.5 Blocking the autofocus 17

5.6 Brightness..... 17

5.7 Contrast 18

5.8 Reading aid..... 18

5.9 Tableau..... 18

5.10 Brake handle..... 18

6 Troubleshooting 18

6.1 Cleaning..... 19

7 Recycling..... 19

8 Warranty 19

9 Technical specifications 20

10 Supplier information 58

11 Manufacturer information 58

This document concerns the original general user manual,
has been prepared by and is commissioned by Rehan Electronics Ltd.
The i-See and i-See HD product names are trademarks of Rehan Electronics Ltd.

Foreword

This user manual is intended for end users as support for the use, installation and maintenance of this device. Read this manual carefully before use and observe all safety instructions to prevent damage to your health, the device or the environment.

The i-See HD has been developed to support reading problems of individuals who have low vision conditions such as macular degeneration, glaucoma, etc.

If you have any questions after reading this manual, please contact your supplier. You will find this information at the end of this user manual.

EN

Important instructions and warnings are placed in black-outlined areas such as this one.

1 Introduction

The i-See HD is a desktop video magnifier of high quality that is user friendly with its physical buttons and turn knobs. It is designed as a desktop appliance that can be used in a home or in an office setting.

You will find a comprehensive overview of options and technical specifications in *Chapter 9 Technical specifications*.

Changes to the device may only be made by the manufacturer or a technician approved by the manufacturer. Unauthorized repairs or modifications may result in injury and will void all warranties.

2 Package contents

Contents of the box:

- i-See HD base
- TFT screen, including control interface
- Power cable
- Instructions for use

3 Safety

- Avoid direct sunlight on the screen and tableau of the i-See HD.
- Be careful when handling the moving parts of the reading table of the i-See HD.
- The i-See HD requires little or no maintenance. If you want to clean the exterior of the device, only use a dry cloth. The use of aggressive cleaning products like alcohol, acetone or other chemical substances is prohibited.
- Do not use the i-See HD as a support for walking, sitting down or getting up.
- Switch off the i-See HD if you do not intend to use it.
- Ensure that the i-See HD is placed in a well ventilated room.
- Place the i-See HD on a stable and horizontal surface to prevent it from falling.
- Ensure that the i-See HD is placed at least 10 cm / 4 inches from the edge of the table.
- Only use the i-See HD indoors. Do not expose the i-See HD to extreme temperature fluctuations or humidity. Do not place the i-See HD next to a heat source or in direct sunlight.
- Only use the supplied cables and connect the power cable to an earthed socket \oplus .
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the i-See HD, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

EN

3.1 Transporting the i-See HD


If the device needs to be transported it is advised to transport it in its original packaging in order to avoid any potential injury or damage to the device. Keep the transport and storage temperature conditions in mind.

4 Installation

The i-See HD will be delivered to you in a pre-assembled state. All you have to do is connect the TFT monitor to the base, attach the cables to the control interface and TFT monitor and connect the power cable to the base.


4.1 Attach TFT screen

You can attach the TFT monitor by hanging it on the base and tightening the bolts on both rear sides of the screen.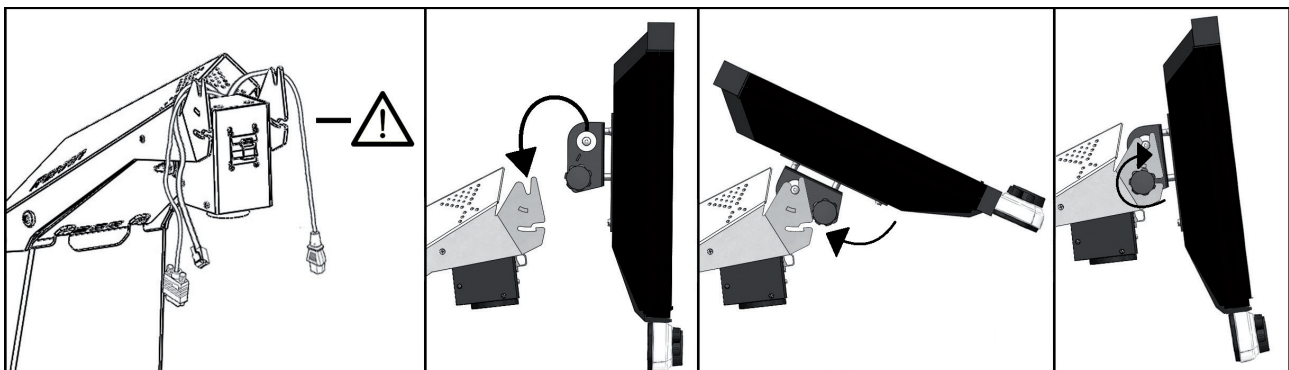


4.2 Connecting the cables

Connect the power and video cables from the base to the monitor. Connect the data cable from the base to the control interface.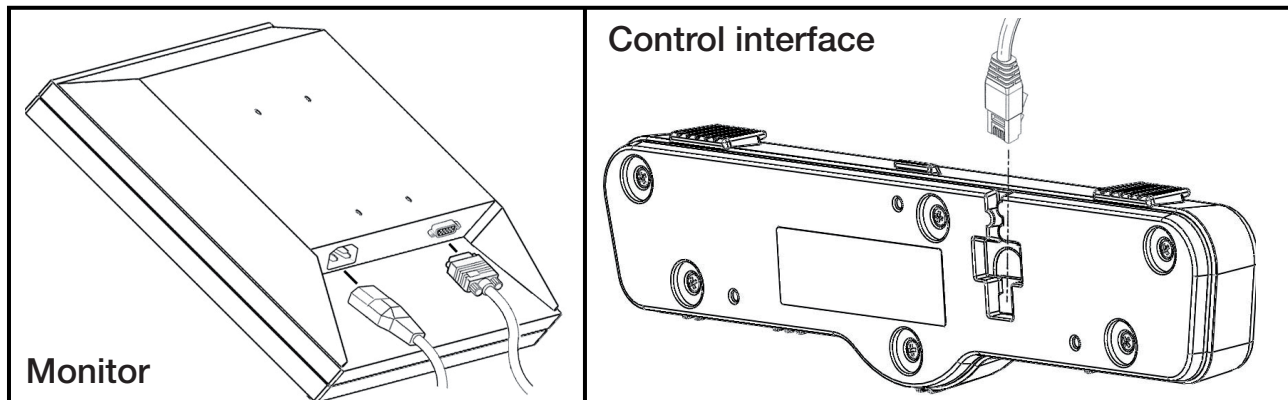


4.3 Tilt the screen

Loosen the bolts at both rear sides of the screen; this loosens up the screen so you can move it in your preferred position. Tighten the bolts to secure the screen at that position.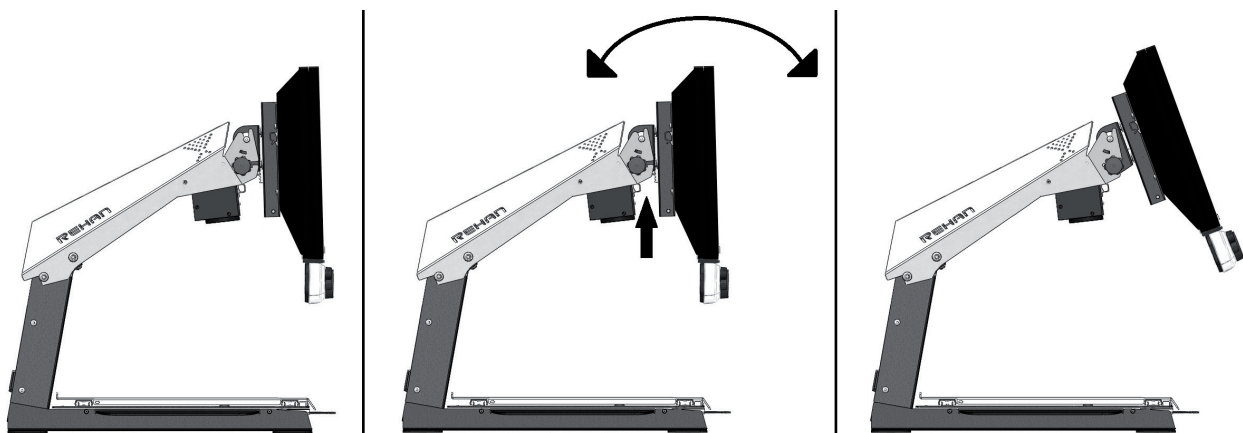

4.4 Adjust the height of the screen

With the i-See HD it is possible to adjust the height of the screen. Simply grab the lever at the right side of the TFT screen, pull it towards you and, while holding your left hand at the lower left corner of the screen, slide the screen up or down. Release the lever to secure the screen at that position.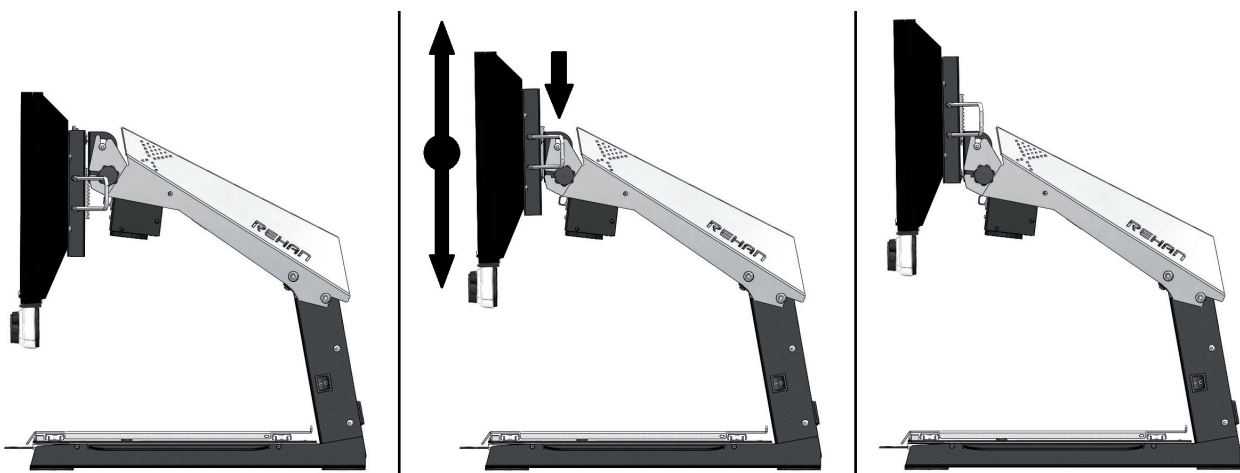


5 Controls

5.1 On/off switch

The on/off switch of the i-See HD is located on the right hand side of the pillar.

The control interface contains all the controls of the i-See HD.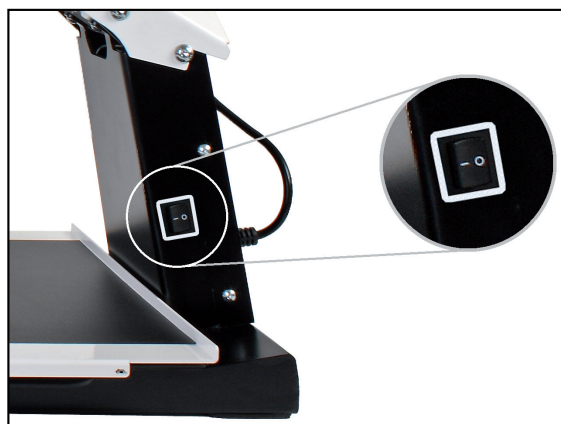


EN

5.2 Display modes

Push the knob to switch between photograph modes or high contrast modes. The last used mode will automatically be selected. Turn the knob to select different display modes.

You can find a detailed overview of the available display modes in the technical specifications section of this manual.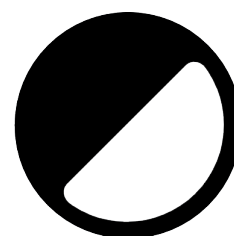


5.3 Zooming in and out

Turn the knob to the right to zoom in and to the left to zoom out.

5.4 Zoom presets

While reading texts and looking at images, you require a zoom setting that suits your needs. The i-See HD contains a function that memorizes these preferred zoom settings for you.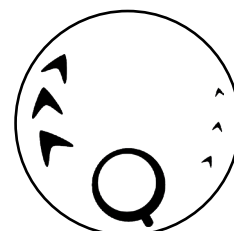

This means that when you switch from the photograph mode to a text mode, the i-See HD will automatically switch to the last zoom setting you used in the text mode. The same goes when you switch from a text mode to the photograph mode – the device will automatically switch to the last zoom setting you used while viewing photos.


5.5 Blocking the autofocus

The i-See HD will always automatically focus on the object on the tableau. However, in certain situations (like during writing), it can be useful to block the autofocus.

Push the zoom knob to block the autofocus. Turn the knob in either direction to unblock the autofocus.

5.6 Brightness

Turn the knob to the left to decrease or to the right to increase the brightness of the image on your screen.

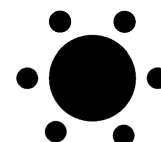5.7 Contrast

In cases where the quality of the paper or ambient lighting condition causes blank spots in high contrast mode, the contrast can be adjusted.

To enable contrast adjustment, press on the Brightness button 3 times to enable the function for 10 seconds. During that time it is possible to adjust the contrast modulation by rotating the brightness knob left or right and improve the image quality. The device will beep once when the function is enabled and when it is automatically disabled.

After powering off the device, the contrast adjustment will revert to its default setting.

5.8 Readings aid

By pushing the knob the user can choose between 4 different reading aids: a horizontal and vertical reading line, a horizontal and a vertical block blank. By subsequently turning the knob you can fine tune your chosen reading aid.

By keeping the knob pressed for 2 seconds, you can immediately switch your chosen reading aid on or off.

5.9 Tableau

The i-See HD contains a tableau on which you can place your reading material. The tableau can move from left to right (X-direction) and from front to back (Y-direction).

5.10 Brake handle

Underneath the tableau of the i-See HD you will find a single brake handle with which you can control the movability of the tableau. The handle has 3 positions:

- Left = all movement of the tableau is blocked
- Middle = the movement from front to back (Y-direction) is blocked
- Right = the tableau can be moved freely

6 Troubleshooting

The i-See HD does not require a high level of maintenance during its expected lifetime. Use the device always within the operating conditions as specified in the technical information.

Problem	Solutions
No picture on screen	<p>Check that the power cable is plugged into mains and that wall socket is switched on.</p> <p>Check that power cable is pushed home and making a good connection.</p> <p>Check that cable from camera is pushed home and making a good connection.</p> <p>Check that the device's on/off switch is set to ON.</p> <p>Check that device LED is lighting.</p> <p>Check that TFT is powered on (the TFT on/off switch is located in the bottom right corner of the TFT).</p>
Blurry picture	<p>Turn zoom knob in either direction to unblock autofocus.</p> <p>Check camera lens is clean. Use dry, soft cloth to clean the lens.</p>
Interface controls not working	<p>Check that cable connecting controller is pushed until it clicks and making a good connection.</p>

If problems persist, then contact your supplier for assistance.

6.1 Cleaning

For safety reasons, always disconnect the AC power adapter from the mains supply before cleaning the i-See HD. Follow the instructions below to help prevent the product exterior from deteriorating or colouration:

- Wipe with a soft, dry cloth.
- Do not use solvents or other chemicals. Do not wipe with a chemically-treated cleaning cloth.
- Use a soft dry cloth to clean all other accessories.

7 Recycling

The i-See HD contains electrical and electronic equipment (WEEE) and should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated collection points where it will usually be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product. Disposing of this product and the packaging correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. If you are unsure, contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

EN

8 Warranty

This product is covered by the warranty provisions of your supplier. For more information about these provisions, contact your supplier.

Never hesitate to contact your supplier or the manufacturer to report unexpected operational incidents. Please note the serial number of the device when contacting the manufacturer.

Manufacturer address
production date

Catalog reference
number

UDI number

Serial number

Device input and
output power

Use earthed socket

Rehan Electronics Ltd.,
Aughrim Road, Carnew,
Co. Wicklow, Ireland

2021-05

REF 2000-11-0029S-22W-U

UDI (01)05391513470653
(21)IS000000

SN IS000000

INPUT 100-240V; 50-60Hz; 1.0A
OUTPUT 1 100-240V; 50-60Hz; 0.8A
OUTPUT 2 --- 13V 1.8A

35°C
5°C

Medical device
purpose and name

Read the
instructions for use

Complies with Part
15 of the FCC Rules

Product conform
E.U. standards

Indoor use
temperature

IP rating

WEEE recycling

9 Technical specifications

Resolution		1920 x 1080	
Video refresh rate		50 Hz or 60 Hz	
Display size		54.6 cm 22 inch	
Display modes <i>These display modes can be turned off in the i-See live mode advanced menu</i>	Photograph	Full colour (or high saturation)	
		Greyscale	
	High contrast	Black - white	Black - yellow
		White - black	Yellow - blue
		Green - black	Blue - yellow
		Black - green	Red - black
		Yellow - black	Black - red
Magnification (With a 22" screen)	Minimum	≈ 3x	
	Maximum	≈ 75x	
Reading Aid	Horizontal Reading Line	Available in all models	
	Vertical Reading Line	Available in all models	
	Horizontal Block Blank	Available in all models	
	Vertical Block Blank	Available in all models	

Power Supply		Input: 100-240VAC, 50/60Hz, 1.0A max. Earth connected	
Power rating		Max: 25W	
Dimensions	22" (screen in highest position)	51 × 48 × 64 cm 20 × 19 × 25.2 inch	
Weight		15.8 kg / 34.8lb	
IP rating		IP20; <i>solid foreign objects</i> ≥ 12.5mm <i>Not protected against ingress of water</i>	
Conditions of use	Temperature	5 to 35 °C 41 to 95 °F	
	Humidity	10 to 85%, no condensation	
Conditions of storage and transport	Temperature	-20 to +60 °C -4 to +140 °F	
	Humidity	0 to 90%, no condensation	

If you have any questions after reading this manual, please contact your supplier.

FCC NOTICE (Class B)

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. this device may not cause harmful interference, and
2. this device must accept any interference received, including interference that may cause undesired operation. Federal Communications Commission Statement.

EMI

The i-See HD complies to harmonised standards according to medical devices and with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/television technician for help.

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